1303 / 1313 NANO

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Intended use:

Spare parts, accessories

Our original spare parts and original accessories correspond to the specifications of our armchair. We therefore advise you only use our original parts. This will ensure a long service life.

Note:

The use of spare parts and/or accessories that do not fit properly or are of inferior quality may damage the armchair, and this is not covered by our warranty.

Safety instructions

- Children, and also persons who have physical, sensory, mental or intellectual impairments and/ or inadequate knowledge, may only use this functional chair if they are supervised or have been instructed on how to use the armchair safely and have understood the risks involved.
- Children are not allowed to play with the armchair.
- When adjusting the setting, it is important to ensure that nobody or nothing can get stuck in between the moving parts. Before operating the armchair, make sure that there isn't anything under the seat.

Proper use

- Only charge the armchair with the charging cable included in the delivery.
- When folded out, the leg support must not be used to lift the functional chair.
- The back and leg support components are not suitable seat surfaces. Maximum load 15kg.
- The functional chair is not suitable for use in damp rooms or in outdoor areas.
- The adjustment function is not suitable for the continuous operation, when the motor is moving without interruption.
- The electrical supply cable and power pack must not be subjected to any crushing, bending or tensile stress.
- The battery pack reaches its optimum performance, when it is used at room temperature under normal conditions (20° – 25°C).
- The power pack must not be covered by objects or textiles (curtains, carpets) as this may lead to it heating up. Risk of fire!
- In the event of faults, visible damage to the components, unusual noises, smells or smoke, disconnect the power pack immediately. Contact the customer service team!
- Liquids must not come into contact with the electrical components. Risk of electric shock! If liquid is accidentally spilled on the parts, disconnect the power pack immediately.
- Do not try to rectify the fault yourself.
- Repairs may only be carried out by customer service. Reconstruction and modifications of original components and original electrical parts are not permitted.

1303 / 1313 NANO

Notes on use:

- To charge the battery, only use the power adaptor provided Match Code: MC120-29 2.0A
- Fully charge the battery before using the battery pack for the first time. This is done by inserting the switching power supply into the wall socket. **See picture «Charging the battery»**
- The LED indicator shows that the power supply unit is connected to the mains.

- As soon as the battery's charge level reaches a critical value, a 'beep' will sound when you use the armchair. This indicates to the user that the battery must be recharged! **See picture «Charging the battery»**
A residual charge is still available to bring the chair into the desired position. However, it is absolutely necessary to charge the battery with the mains adapter supplied.
- It is possible to charge and adjust at the same time.
- When the battery is fully charged and plugged into the mains, the mains adaptor will continue to supply power.
- It takes approx. 5–8 hours to charge the battery.
- Thanks to the integrated self-protection mechanism, the armchair electronics are switched off when overloaded and it stops in the current position. The electronic armchair is functional again after a few seconds and responds to switch commands.
- To replace the battery, follow the **procedure «Charging the battery»** in this manual.

1303 / 1313 NANO

Li-ion battery

- Please read all safety instructions and manuals. Failure to comply with safety instructions and manuals may lead to serious injuries.
- Keep all safety instructions and manuals for future reference.
- Protect the battery against heat, e.g. continuous sunlight, fire, water and moisture. There is an explosion hazard.
- Fumes may be released if the battery is damaged or not used properly. Breathe in fresh air and consult a doctor if you have any symptoms. Fumes can irritate the respiratory tract.
- If use incorrectly, liquid may leak out of the battery. Do not touch it. If you accidentally come into contact with it, rinse off with water. If the liquid gets into your eyes, seek medical attention. Contact with battery fluid can cause skin irritations or burns.

Instructions for disposal:

Packaging

- The packaging protects the armchair during transport. It is made from recyclable materials that can be re-used. Dispose of packaging materials that are no longer needed at the regional collection points.

Battery

- Do not throw the battery away with the household waste!
The armchair is fitted with a rechargeable lithium-ion battery. The battery can be returned to an authorised stockist or at a corresponding regional collection point. This ensures it is disposed of properly.

Specifications

Weight:	53 to 67 kg
Power:	230V/50Hz
Connection fitting:	Europlug

Customer Service

Order number for Spare Battery: 1303.999

If you have any questions, please do not hesitate to contact your stockist or our customer service team.

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Plug the charging cable into an electrical socket and fully extend the height setting to access the battery.

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The battery is on the transverse joint at the back, underneath the seat.

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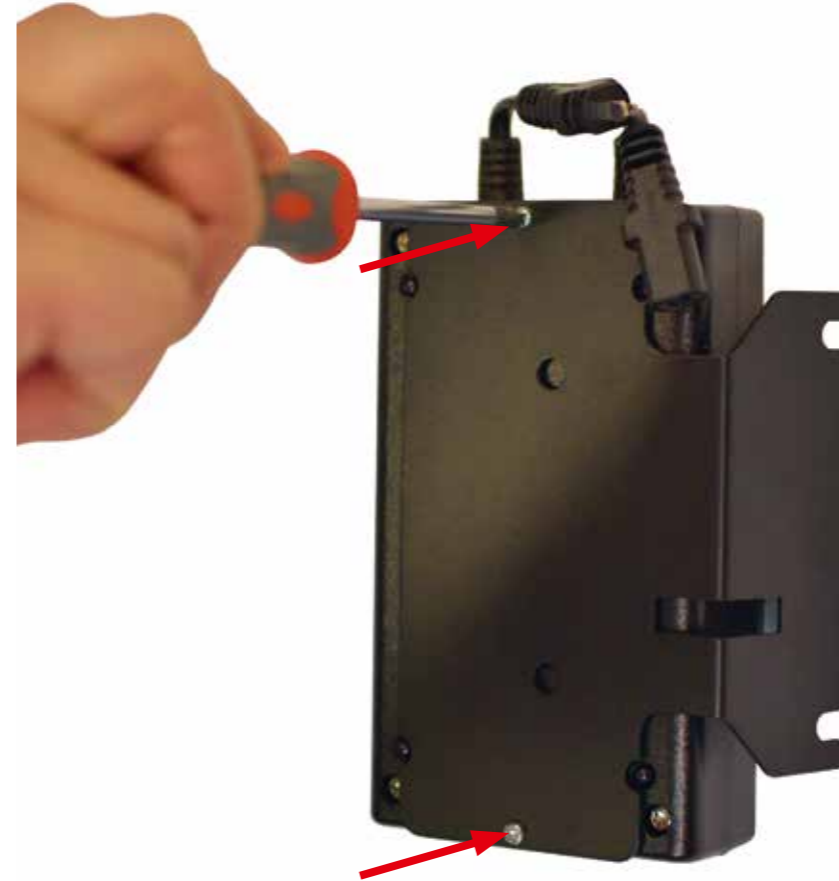
Slightly loosen the battery holder using a spanner wrench SW 10, then remove the battery along with its holder.

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Unscrew both cross-head screws on the battery unit. Take the holder off the battery and screw it onto the new battery.

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Attach the battery holder back onto the frame and tighten both hex-head screws again.

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Connect the cable again and check that all the functions work.

WEIBELWEIBEL AG

Intertime pure sitting
Brühlstrasse 21
5304 Endingen
Schweiz

T +41 56 265 88 88
info@weibelweibel.ch
intertime.ch

Intertime
Switzerland